PRESENTATIONS:

1 Syringe of 3 g and 3 tips. Available in colors: Clear, A1, A2 and A3.

DESCRIPTION:

Product based on monomers and fillers used in the relining of bruxism plates and 3D printed provisional crowns. Due to its consistency, the material facilitates sculpting and modeling in techniques that require application of large amounts of product.

INDICATION:

PriZma 3D Rebase is a product indicated for use in the relining of 3D printed bruxism plates and provisional crowns.

COMPOSITION:

Methacrylic monomers, silica, initiators and co-initiators.

MODE OF USE:**1. Plates Rebasings:**

- Place PriZma 3D Rebase resin in small quantities into the myorelaxation plate or temporary crowns where adaptation and retention are to be improved. Insert into patient's mouth without curing, fully seat, light cure (blue light) for 30 seconds. Remove from the mouth, finish by removing excess material and cure outside of the mouth for an additional 30 seconds. During light curing, the patient may feel a slight warming due to the reaction, but this will depend on the sensitivity of each patient.

2. Finish and polishing:

- After light curing complete, sculpt the piece. Carry out the finishing in the conventional way. It is recommended to initially finish using a tungsten carbide bur and then use polishing cutters to even out the surface.
- Clean with neutral detergent and plenty of water. Dry the entire surface using oil-free compressed air.

Note 01. In the relining procedure, it is advisable to clean with neutral detergent and plenty of water to remove possible remains of monomers fixed to the material.

Note 02. No significant color change is expected during the use of the product, however, follow-up should be performed frequently with the dental surgeon. If the material presents significant color change, to the point of compromising the aesthetics, it is recommended that new devices be made.

WARNINGS:

- Exclusive use of the dentist surgeon.
- Do not use the product after its expiration date.
- Do not reuse empty packaging.
- Keep the material out of the reach of children.
- This material was manufactured for dental use only and must be handled according to the instructions for use. The manufacturer is not responsible for

damages caused by other uses or by incorrect handling not described in this manual. Furthermore, the user is obliged to verify, prior to use and under his/her own responsibility, whether this material is compatible with the intended use, especially when this use is not indicated in the present directions for use.

- PriZma 3D Rebase must not be ingested or aspirated. In case of ingestion or aspiration, seek medical attention immediately.

PRECAUTIONS:

- Wear PPE when handling the product, including gloves and protection equipment. It is recommended that patients also wear protective eyewear.
- Do not allow loose polymerized objects and parts to be swallowed or aspirated.
- Contact of unpolymerized PriZma 3D Rebase with skin, mucous membranes and eyes should be avoided. When not yet polymerized, the product can have a slight irritating effect.
- If the product meets the hands, wash with plenty of water.
- In case of allergic reactions, discontinue use immediately. If necessary, seek medical advice.
- Single use product.
- Reprocessing prohibited.
- Do not expose the material to temperatures above 25 °C.
- Do not expose material directly to sunlight.
- Replace cap on syringe immediately after use.

CONTRAINDICATIONS:

Do not use this material on patients who are allergic or hypersensitive to any component of the formula.

ADVERSE EFFECT:

Until now no adverse effects of the product have been identified.

STORAGE CONDITIONS:

- Store the product in its original packaging and in a clean and dry place.
- Conserve PriZma 3D Rebase properly sealed and without exposure to light or high humidity.
- The product should be kept at a controlled temperature between 4 °C and 25 °C.

VALIDITY:

If properly stored, the product is valid for 24 months from the date of manufacture. Do not use after expiration date.

DISPOSAL:

Do not reuse the product and dispose of it according to local legislation.

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