DESCRIPTION:
PriZma Aligner Fill fluid resin is a light-curing filler for pontics in thermoplastic aligners.

PRESENTATIONS:
• 1 syringe of 2 grams + 5 application tips.
Available in the following colors: BL1, BL2, BL3, B1, B2, B3, A1, A2, A3, C1, C2 and C3.

INDICATION:
PriZma Aligner Fill fluid resin is intended to be used as a light-curing filler for pontics in thermoplastic aligners. The material has the ideal consistency for application directly to the pontic and, after the light-curing process, acquires suitable flexibility characteristics that do not interfere with the aligner's movements.

COMPOSITION:
Monomers, fillers, initiators and pigments.

- MODE OF USE:**
1. Thoroughly clean and wash the inner surface of the pontic to be filled.
 2. Apply an adhesive to the pontic, polymerize for the necessary time according to the manufacturer of the medical device and proceed with the application of PriZma Aligner Fill fluid resin in the desired shade.
 3. Open the syringe and insert the applicator tip. Apply PriZma Aligner Fill directly to the pontic area. Start with the buccal side and light-cure each layer for 40 seconds with a blue light device of at least 1200mW/cm². A microbrush moistened in the adhesive can be used to make accommodations.
 4. Apply the material until the pontic is completely filled, or at least 3 layers of material.
 5. After applying the final layer of PriZma Aligner Fill, light-cure for 40 seconds on the opposite side of the aligner, starting from the buccal and lingual surfaces.
 6. Check that the fit of the aligner used by the patient has not been compromised. If the socket has been deformed after application of the flowable resin, the PriZma Aligner Fill should be cut out or removed and replaced.
 7. The applicator tips are single-use devices. Applicators should be discarded and replaced between applications during the procedure.

- WARNINGS:**
- Use the indicated curing time (40 seconds). If necessary, depending on the power of the curing light, increase the curing time. In all cases, pay attention to any signs of patient discomfort due to the exothermic reaction (slight heating) during the polymerization process.
 - The product is for professional use only.
 - Do not leave the product within the reach of children.
 - Do not reuse empty packaging.
 - Do not use the product on patients who are sensitive to the components in the formulation.
 - Do not use the product after its expiration date.
 - This material has been manufactured for dental use only and must be handled in accordance with the instructions for use. The manufacturer is not responsible for damage caused by other uses or by incorrect handling not provided for in this document. In addition, the user is obliged to check, before use and under their own responsibility, that this material is compatible with the intended use, especially when this use is not indicated in this instruction for use.
 - Do not expose the material to temperatures above 25 °C.
 - Do not expose the material directly to sunlight.
 - Do not allow loose polymerized objects and parts to be swallowed or aspirated.
 - Do not use the material if the packaging is damaged.
 - In the event of an allergic reaction, discontinue use immediately. If necessary, seek medical advice.
 - Do not store PriZma Aligner Fill next to

eugenol-based products.

- PRECAUTIONS:**
- Wear PPE when handling the product, including gloves and goggles. It is recommended that the patient also wears goggles.
 - Eye protection should be worn when using a light-curing device.
 - PriZma Aligner Fill is contraindicated when it is not possible to establish a dry operative field or correctly follow the technique described.
 - Single-use product. After polymerization it does not return to its original state. Reprocessing is prohibited.
 - In case of contact with eyes, rinse thoroughly with running water and contact a doctor immediately.

- CONTRAINDICATIONS:**
- Avoid using PriZma Aligner Fill in patients with a history of allergy to acrylate-based composite resins.
 - PriZma Aligner Fill is contraindicated in patients who are allergic or hypersensitive to methacrylates and related monomers and should not be used in these patients. In the event of an allergy, discontinue use and ask the patient to seek medical attention.
 - To avoid allergic reactions to the methacrylate monomer, it is recommended that gloves are worn at all times.
 - The product should be discarded immediately if PriZma Aligner Fill detaches from the pontic. Do not reintroduce product that has already been light-cured into the pontic.
 - If PriZma Aligner Fill comes into contact with your hands, rinse with plenty of water.
 - PriZma Aligner Fill should not be swallowed or inhaled. If swallowed or inhaled, seek medical attention immediately.

ADVERSE EFFECT:
To date, no adverse effects of PriZma Aligner Fill have been identified. However, there are reports in the literature of allergic reactions involving dental products based on methacrylic monomers.

STORAGE CONDITIONS:
Store PriZma Aligner Fill in its original packaging in a clean, dry place. Keep the material properly sealed and away from light or high humidity. The product should be kept at a controlled temperature between 4 °C and 25 °C.

VALIDITY:
If stored according to the manufacturer's instructions, the product is valid for 24 months from the date of manufacture.

DISPOSAL:
Do not reuse the product and dispose of it in accordance with local legislation.

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